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# Zephyr<sup>®</sup> by Dry & Store<sup>®</sup>

## Physical Properties

- Outside dimensions are 150 mm x 92 mm x 72 mm (5.9" x 3.6" x 2.8").
- The drying tray has two compartments: the forward drying area measures 80 mm x 65 mm x 24 mm (3.2" x 2.6" x 1.0"), and a small Dry-Brik<sup>®</sup> compartment is in the rear of the tray.
- Weight as shipped is approximately 582 g (1 lb, 4 oz ).

## Power Supply

Input: 100-240 volts AC, 50/60 Hz, 1.0A max. Output: 24 VDC at 15W max. Zephyr power supplies are interchangeable with Dry & Store<sup>®</sup> Global and are CE Marked. They are certified to ETL, UL, and CSA standards, and carry GS (Germany), PSE/JET (Japan), C-Tick (Australia/New Zealand), and SABS (South Africa) certifications.

## Operating Features

- A ball-bearing fan continuously circulates warm, dry air for the entire operating cycle, causing moisture to move out of the hearing instruments by diffusion. A desiccant in the airflow path strips the moisture out of the air, ensuring continuous dry operation.
- Zephyr will dry cerumen, making it easier to remove from the hearing instrument.
- Zephyr turns off automatically after 8 hours unless the ON/OFF button on the side is pressed.
- Batteries do not have to be removed from hearing devices.
- Zephyr is not designed to diagnose, prevent, monitor, treat, or alleviate disease.

## Desiccant

A disposable Dry-Brik<sup>®</sup>, which must be replaced every two months, is included with each Zephyr. (See Preparations for Use in the operating instructions.)

## Rechargeable Canister Desiccant (optional)

A small number of international configurations are shipped with an optional metal canister desiccant instead of a Dry-Brik. The protective label should be removed from both sides before use. As moisture is adsorbed by the desiccant, the indicator will change from orange (good) to green (bad), or from blue (good) to light pink (bad), depending on the model. The color change signals that the desiccant can no longer adsorb moisture. To restore function, place the metal can in a conventional oven for a minimum of 3 hours at a temperature of 105°-150°C (225°-300°F). The color will change back to orange or blue.

**Warning: Never place metal in a microwave oven! After recharging, allow the hot metal can to cool before use. Best results will occur if the can is covered by a glass during the cooling period.**

**Zephyr by Dry & Store® is protected by U.S. Patents D536,491 and 5,852,879.**

Ear Technology Corporation  
P.O. Box 1017  
Johnson City, TN 37605-1017  
USA  
[info@dryandstore.com](mailto:info@dryandstore.com)  
[www.dryandstore.com](http://www.dryandstore.com)

© Ear Technology Corporation, 2009  
Group 3, REV E 6/09